Momini

TwinHarmony

Create happy beginnings

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01. SECURITY:

Before using the product, read this manual and follow the guidelines during operation.

02. DEVICE DESCRIPTION - 2 SETS [PAGE 2 - A]:

1. Device / 2. Cylinder (silicone) / 3. Valve (silicone) / 4. Pump body (polypropylene) / 5. Flange insert 21mm (silicone) / 6. Flange insert 24mm (silicone) / 7. Milk container cap (polypropylene) / 8. Silicone ring for milk container (silicone) / 9. Milk container (nylon polyethylene) / 10. Charging cable

WARNING [PAGE 2 - B]: Measure your nipple size and choose the appropriate flange insert. The box contains 21mm/24mm flange inserts.

03. ASSEMBLY OF THE PUMP [PAGE 2 - C]:

Wash your hands and ensure the device is clean.

1. Place the cylinder on the pump body.
2. Insert the pump body into the device.
3. Insert the valve into the milk container cap. Place the silicone ring on the bottom of the cap. Attach the cap to the container and turn until it clicks.
4. Insert the milk container into the bottom of the device until you hear a click.
5. Do not exceed the maximum milk container capacity. If needed, turn off the pump and empty or replace the container.

04. DISASSEMBLY OF THE PUMP [PAGE 2 - D]:

1. Press the button on the device and disconnect the milk container.
2. Remove the pump body from the device.
3. If you don't need to feed the baby immediately, attach the cap to the milk container and store it in the fridge. Wipe the device with a dry towel. Wash and dry the remaining parts.

05. CHARGING [PAGE 2 - E]:

Fully charge the product before first use. To charge the pump, insert the small plug into the Type-C socket on the device and connect the USB port to a computer or adapter (5V/1A). The device must be off while charging. A flashing indicator means the battery is charging, a steady light means the device is fully charged. Then disconnect from the power source.

06. BUTTON DESCRIPTION [PAGE 2 - F]:

1. Long press to turn on/off the device. Short press to stop. The pump will automatically turn off after 30 minutes of operation or 10 minutes of pause/inactivity.
2. Press to increase or decrease suction level (from 1 to 9).
3. Press briefly to change phases:
4. Sustained expression phase: 5 light, 1 strong suck

5. Massage phase: 9 power levels for breast massage before milk expression

6. Expression phase: constant frequency and strength. It has a gentle sucking rhythm.

7. Simulation phase: two short, one long, three short, two long sucks.

07. DEVICE USER MANUAL [PAGE 2 - G]:

1. Make sure all parts of the breast pump are clean/sterilized and that the device has been assembled correctly (follow the instructions in this user manual)
2. Wash your hands and breasts.
3. Sit up straight and lean forward slightly. Maintain good posture while pumping. DO NOT lie on your side, you cannot tilt the breast pump to avoid milk backflow.
4. Position the center of the collar over your nipple and press it against your breast to ensure regular latching and prevent air from entering.
5. Long press the start button to adjust the suction phase and level according to your preferences.

NOTE: Do not fill the container with too much milk, do not exceed the maximum capacity of 120 ml. In this case, turn off the device, replace the container or pour the milk into another container.

08. STORAGE AND WARMING MILK:

1. You can store expressed milk in the fridge for 48 hours at around 4°C. When freezing, leave space at the top of the container/bottle.
2. To preserve the nutrients in expressed milk, you can heat it in water at 40°C or use a bottle warmer.
3. Do not defrost frozen milk in a microwave or boiling water (it will damage its nutritional value).

09. CLEANING AND STERILIZATION:

1. The device and charging cable cannot be washed or disinfected. If the surface is dirty, clean it with a soft cloth.
2. The milk container cap, container, pump body: made of plastic, can be washed in water or dishwasher (temperature must not exceed 100°C). After cleaning, they can be disinfected using a steam or ultraviolet sterilizer, microwave (place parts in a sterilization container or bag), or boiling water (water level must be higher than the parts to be disinfected, and elements must not touch the bottom or metal edges).
3. The cylinder, valve, flange insert, milk container ring are made of silicone. Due to the delicate material, clean and rinse them gently with water. Do not rub or disinfect them at high temperatures for a long time. Otherwise, they may age or deform quickly.
4. After cleaning and/or sterilization, the device parts must be dried before use.

10. WARNINGS:

1. All parts must be dry before using the device to prevent motor damage by water.

2. Charge the battery according to the instructions. The battery can only be replaced by authorized services, and using an incorrect battery model can cause an explosion. Dispose of used batteries according to local regulations.

3. If the product is not used for a long time, the battery will discharge. Connect it to power and charge it.
4. The pump motor and other electrical parts CANNOT be washed in water or sterilized in a steam sterilizer.
5. Please use the charging cable provided with this product. Do not connect a different manufacturer's cable to avoid damaging the device.
6. Choose an adapter with AC input voltage of AC100-240V and DC output of 5V and 1A.
7. This device MUST NOT be used by children and persons with physical, sensory, or mental limitations or lack of experience and knowledge unless supervised or instructed on safe use and understand the hazards. It is not a toy; children should not play with the device.
8. For lactation problems or pain, consult a doctor before using the pump.
9. Store in a dry and cool place away from children.

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[EN] Rules for handling electrical and electronic equipment: Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Appropriate household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m² devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005.

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[EN] Short declaration of conformity: The manufacturer declares, under its sole responsibility, that the TwinHarmony device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility, Directive 2014/53/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to the making available on the market of electrical equipment intended for use within specified voltage limits, Regulation (EC) No 1935/2004 of the European Parliament and of the Council of 27 October 2004 on materials and articles intended to come into contact with food and repealing Directives 80/590/EEC and 89/109/EEC, Commission Regulation (EU) No 10/2011 of 14 January 2011 on plastic materials and articles intended to come into contact with food, together with Commission Regulation (EU) 2020/1245 of 2 September 2020 amending and correction of Regulation (EU) No 10/2011 on plastic materials and articles intended to come into contact with food, and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in equipment electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances and Directive of the European Parliament and of the Council (EU) 2017/2102 of 15 November 2017 amending Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment. The full content of the declaration of conformity is available at www.momini.eu/deklaracje

Importer: Beauty Factor Sp. z o.o. Plac Bankowy 2, 00-095 Warszawa, Polska
Model: TwinHarmony

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EN | More information about the product is available on the manufacturer's website at:

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