NanoFlush

Create happy beginnings

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01. SECURITY:

Before using the product, read this user manual and follow the recommendations contained therein when using the device.

02. DEVICE DESCRIPTION [PAGE 2 - A]:

1. Mode indicator.
2. Charging port.
3. Power button.
4. Battery indicator.

03. DESCRIPTION OF PARTS [PAGE 2 - B]:

1. Cover.
2. Device.
3. Charging cable.
4. Cleaning brush.
5. Nasal rinsing nozzle.
6. Lid.
7. Used water compartment.
8. Suction tube.
9. Brine solution chamber.
10. Gourd nozzle.
11. Funnel nozzle.
12. Container.
13. Lid.
14. Collection cup.

04. BEFORE USE:

Make sure your device is fully charged before first use by plugging the USB cable into the charging port and charging your device for the first time. This device cannot be used while charging.

05. INSTRUCTIONS FOR USE:

1. Sterilization and disinfection
The nasal irrigator is a reusable device that must be cleaned and disinfected before first use.
2. Rinse the nozzle tip and water tank under running water to remove any dirt.
3. After rinsing, if there are dry stains, soak the stain with cleaning agent and then wipe it with a clean cloth.
4. After the second step, rinse the device again with running water.
5. After cleaning, immerse in purified water at 90°C for 5 minutes, then dry and use.

Note: Do not soak in boiling water. Do not disinfect at too high a temperature or for too long a time.

06. HEAD ASSEMBLY [PAGE 2 - C -1]:

Follow the steps below to assemble the wash head and do not skip any parts.

07. GUIDELINES FOR USING THE DEVICE [PAGE 2 - D]:

1. Place the sealing cap on the collection cup.
2. Follow the direction of the arrow and screw the container to the base of the collection cup.
3. Select the nozzle you want to use and make sure the silicone part fits the container exactly.
4. The funnel nozzle is recommended for cleaning runny noses or mucus, and the gourd nozzle for cleaning dried runny noses.
5. Follow the direction of the arrow and screw the aspirator head onto the main unit by turning clockwise.
6. Press the power button to start the device and adjust the modes by pressing the power button again. The default mode is "Soft Mode".
7. Place the suction tip in the nasal space to clear a runny nose.
8. After use, turn off the device, unscrew the parts counterclockwise to disassemble them, pour out the remaining liquids and waste water from the tank, then pour about 20ml of clean water into the tank, turn on the device and wait until the water is used. Then wash and reassemble for next use.

Tip: If your baby's nose is dry, first soften it with saline and then use the device.

08. ADDING THE SOLUTION TO YOUR NOSE:

Pour the nasal rinsing solution or saline into the solution chamber, taking care not to exceed the maximum volume of 20 ml.

09. PRECAUTIONS:

1. Do not store in a place where it may be exposed to water.
2. Do not store in a place that will be exposed to harmful factors such as humidity, sunlight, sea breeze, sulfur, heat sources, etc.
3. Avoid tilting, vibration and impact.
4. Avoid direct influence of rain and snow, do not place the device together with corrosive substances.
5. In people with a stuffy nose, the pressure from the nasal irrigator may cause excessive pressure in the nasal cavity and middle ear, which may lead to discomfort.
6. If you experience severe middle ear discomfort during or after using the device, stop irrigation immediately and seek further advice from your doctor.
7. Do not blow your nose immediately after using the device, instead gently squeeze the water from your nose with a tissue, otherwise you may experience discomfort.

8. Do not use the device continuously for more than 5 minutes; turn off the device once you have finished washing your nose and the waste water tank has been emptied.
9. The use of the nasal irrigator is not recommended for children under 5 years of age. Children who are 5 years or older must use the device under adult supervision.
10. Adults should guide children on how to use the device properly and safely.
11. Patients with recent surgery should consult a doctor before using the device.
12. Check whether the surface of the product is undamaged.
13. Check that the detectors and plugs are in place and that the power cable can be connected to the socket.
14. Do not use the product while bathing.
15. Do not use the appliance if it has a damaged cable or plug, is not working properly, or if the cable or plug connection is broken.
16. Do not direct the water jet at eyes, ears or other sensitive areas of the body.

● **Material:** PC/ABS/PP/Silicone
Brine chamber: 20±5ml
Maximum pressure: ≤ 0.63 Mpa
Input power: DC 5V 1A
Battery: 3.7V, 950mAh
Charging Port: USB C
Cable length: 1.2m
Working power: ≤ 3W

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[EN] Rules for handling electrical and electronic equipment: Waste electrical and electronic equipment as well as batteries and accumulators cannot be mixed with other waste. This is indicated by the crossed out bin symbol on the product packaging. Electrical and electronic equipment as well as batteries and accumulators may contain hazardous substances, mixtures and components harmful to the environment and human health. Improper handling or damage may result in damage to health or environmental pollution. Proper household handling of waste electrical and electronic equipment and batteries and accumulators positively contributes to the reuse and recovery, including recycling, of waste equipment. Waste electrical and electronic equipment as well as batteries and accumulators can be returned to local selective waste collection points. The distributor is obliged to collect used equipment from households free of charge at the point of sale, provided that the used equipment is of the same type and performed the same functions as the equipment sold. When delivering equipment intended for households to the buyer, the distributor is obliged to collect used equipment from households free of charge at the place of delivery of the equipment, provided that the used equipment is of the same type and performed the same functions as the delivered equipment. A distributor running a retail unit with a sales area within the meaning of Art. 2 point 19 of the Act of 27 March 2003 on spatial planning and development of at least 400 m² devoted to the sale of equipment intended for households, is obliged to accept waste equipment from households, free of charge, in this unit or in its immediate vicinity, none of the external dimensions exceeds 25 cm, without the need to purchase new household equipment. The product was introduced to the market on August 13, 2005 r.

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[EN] Short declaration of conformity: The manufacturer declares, under its sole responsibility, that the device complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonization of the laws of the Member States relating to electromagnetic compatibility and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Directive Commission delegated act (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances. The full content of the declaration of conformity is available at www.momini.eu/deklaracje

Importer: Beauty Factor Sp. z o.o. Plac Bankowy 2, 00-095 Warszawa, Polska
Model: NanoFlash

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EN More information about the product is available on the manufacturer's website at:

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