neno

sinus

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neno
www.neno.pl
inspired by Children,
designed by Parents

BY **Xblitz**

neno

sinus

USER'S MANUAL

Dear Customer,
the Neno Sinus product that you have purchased is an electric aspirator, used to suck the secretions out of a child's nose. The device is made to be used on newborns and children over the age of one.
Please read the instructions before using the device.

1. PRECAUTIONS

- The product is not a toy – don't let children play with it.
- Test the suction force on the back of an adult's hand before use.
- Use the USB cable included with the device to make sure that the charging voltage matches the device's parameters. Do not connect the device to a voltage higher than that specified in the device specification - it may damage the device and cause a fire or an explosion.
- Do not charge the device during use.
- If you will not be using the device for a long period of time (at least 3 months), fully charge the device before storing it.
- Do not disassemble the device or repair it yourself. In case of defects, contact the service center.
- Do not use the device if you can see lacerations or symptoms of inflammation in the baby's nose.
- In case of fever, infection, irritation or bleeding nose, stop using the device and contact your doctor.
- This aspirator is a hygiene product. It is only used to clean naturally formed secretions from the baby's nose. The device should not be used if the child is ill.

2. PRODUCT CHARACTERISTICS

- Replaceable silicone tips:**
The four types of tips included allow you to adjust the device to your child's age and needs.
- Three suction modes:**
Soft, moderate and strong suction modes between which you can freely switch to suit your child's needs.
- Built-in timer:**
The device turns off automatically after three minutes of use to prevent unnecessary battery drain.
- Safe design:**
The device being bent at a 90-degree angle makes it safe to use and it is not possible to push it too deep into the baby's nose.
- Lighting:**
The additional, separately activated LED light on the device allows you to use the aspirator even at night. The gentle, warm light of the lamp does not irritate the eyes or skin of the child and allows you to see the inside of the nose more easily.
- Easy to clean and waterproof:**
The aspirator is IPX6 waterproof, so it can be washed under running water. Please do not submerge the device underwater.
The tips and the secretion container can be removed from the machine, allowing the aspirator to be washed more thoroughly, especially when it comes to residue.
- Small and portable:**
The device is lightweight and easy to use thanks to its convenient handle and large buttons. Thanks to the built-in battery, the device is portable.
- High-end battery:**
The device has a built-in 400mAh lithium battery that can be charged via a micro USB port.

3. DEVICE DESCRIPTION

4. PACKAGE CONTENTS

- Silicone gourd-shaped tip (flat)
- Silicone gourd-shaped tip (oblique)
- Silicone funnel-shaped tip (flat)
- Silicone funnel-shaped tip (oblique)
- Secretion container
- Anti-reflux container lid
- Outer lid of the container
- USB cable (for charging)

5. INSTALLATION

- Place the anti-reflux lid on the container.
- Place the container with the lid in the device.
- Place the outer lid of the container on the device and tighten it in place, rotating it clockwise to make sure it stays in place and the device works properly.
- Select the appropriate silicone tip and place it on the outer lid of the container.

Make sure that all items are placed in the correct order during installation and that the outer lid is screwed clockwise. When assembling the device, make sure that the seals are properly placed on the container and housing of the unit. If one of the seals is not on the correct part of the aspirator, the suction function of the device will be affected.

6. SILICONE TIPS

- Gourd-shaped tips (flat, oblique):** they are used to clean the noses of newborns. The shorter tips are better suited for the noses of newborn babies, and the bubble gourd of the tip blocks the nostril, making it easier to suck out the secretion. The oblique tip can be used to clean the sides of the nostrils.
- Funnel-shaped tips (flat, oblique):** used to clean the nose of children that are over 1 year old. The funnel-shaped tip goes deeper into the nose, making it better suited for use with children who have already grown up. The oblique tip can be used to clean the sides of the nostrils.

7. HOW TO USE

- Assemble the device according to the instructions in the "Installation" section. Make sure that the seal is placed on the aspirator housing and that a suitable silicone tip is placed on the outer lid of the container. Select the suction mode that best fits the needs of your child. **Specific situations:**
 - If the discharge in the baby's nose is dry, soften it with saline solution before using an electric aspirator.
 - If the discharge is thin and watery, you can directly use an aspirator to extract it.
 - If you do not see secretions but the child "wheezes" when breathing through their nose, it is an indication of a stuffy nose. It is recommended to gently stimulate the nose so that the baby sneezes and suck the discharged secretion with the aspirator.
- Recommended method of use:**
Before using the aspirator on your child, test the strength of the device on back of your hand and adjust the mode of operation of the device to the level that seems to be the most suitable. **Charge the device for two hours before first use to ensure a longer battery life.** Power button:
 - press – soft suction mode
 - presses – moderate suction mode
 - presses – strong suction modePressing the power button when strong suction mode is on turns the device off.

8. CLEANING AND MAINTENANCE OF THE DEVICE

- The tips and the container can be removed from the device and washed separately. They can also be rinsed under running water and cleaned with the use of a brush.
- You can use a cotton swab dipped in medical alcohol to clean the air outlet.
- Make sure that all parts of the device are dry before reassembly to prevent bacterial contamination.
- Do not use abrasives or corrosive cleaning products when cleaning the machine. This can negatively affect its lifespan.
- Turn off the device during cleaning.
- Do not submerge the device under water.

9. TECHNICAL SPECIFICATION


Output voltage:	5 V
Operating voltage:	3,7 V
Power:	2,2 W
Battery:	3,7 V Lithium, 400 mAh
Waterproof:	IPX6
Maximum negative pressure:	<-60 kPa
Charging time:	approx. 1,5 h
Operating time:	approx. 1 h
Weight:	153 g
Dimensions:	154x32x70 mm (without silicone tip)

10. WARRANTY CARD

The product is covered by a 24-month warranty. The terms of the guarantee can be found at: <https://neno.pl/warranty>

Details, contact and website address can be found at: <https://neno.pl/contact>

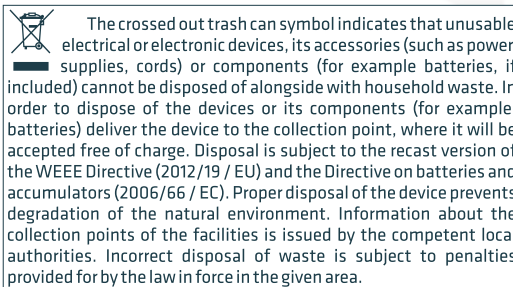Specifications and contents of the kit are subject to change without notice. We apologise for any inconvenience.

Manufacturer:
KGK Trade sp. z o.o. sp. k.
Os. Urocz 12
31-953 Cracow
Poland
Made in P.R.C.

neno
www.neno.pl
inspired by Children,
designed by Parents

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