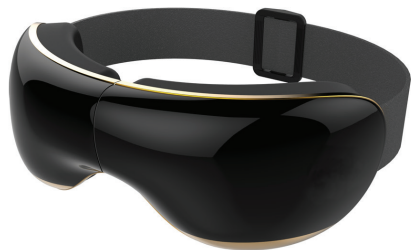
medivon.

**User manual**  
EYE MASSAGER

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## 1. Opis produktu

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## 1. Product description

The Horizon Travel eye massager offers a vibration massage, hot compress and regenerating massage at acupuncture points around the eyes. These treatments relieve tired eyes and relax the user both physically and mentally. The product has relaxing music pre-loaded, which increases the comfort of the massage. The device looks like glasses, is fashionable and practical, is easy to carry and has a built-in rechargeable battery. The integrated and compact device will serve everyone, it can be used anytime and anywhere, whenever you want. Allow yourself and your eyes to relax.

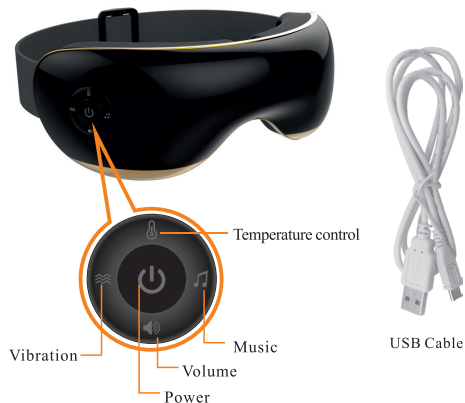
## 2. Application

The product has a health function, it is suitable for adolescents with pseudomyopia. The product is aimed at students, people who strain their eyesight at work and work long at the computer, writers, accountants, people sitting long in front of a computer or TV screen, the elderly experiencing eye fatigue, people who often stay awake late or otherwise abuse their eyesight.

## 3. Properties

- Vibration massage: massage simulation (vibration, churning, pressure, rolling and other traditional techniques) at all important acupuncture points in front of the eyes.
- Hot compress: increasing the temperature of the device accelerates blood circulation around the eyes, bringing relief and relaxation.
- Heating and vibration: combination of heating and massage of the acupuncture points of the eyes.
- Silicone elements increase the comfort of use.
- Built-in battery, easy to charge anywhere.
- 15 minute work cycle.
- The ability to play music

## 4. Product appearance

## 5. Operation

1. Put on the massager correctly, short press the power button, the massager will start automatically with the functions: vibration, heating and music play.
2. After booting, briefly press the power key to switch between different modes.
3. The music will play automatically, press and hold the music key for 3 seconds to stop playing the music.

4. The heating function will start automatically, press and hold the temperature button for 3 seconds to turn off the heating function.
5. Music volume can be adjusted while the massager is working, there are four volume levels.
6. The massager will automatically stop working after 15 minutes.
7. The recommended length of massage is 15 minutes (one cycle) at a time, if it is necessary to use it again, press the power button again.
8. Holding the power button for 3 seconds will make the device turn off automatically.

## 6. Function keys

1. "🔌" Power Button: Press the button shortly to start the massager, it will turn on to the default mode and green backlight. After activation, the button will act as a function button for changing the mode - each short press will change the operating mode. Long press of this button will turn off the massager.
2. "🎵" Music key: Use it to change the currently playing track to the next one. There are six songs built into the device, the first one will be played by default. Press and hold the button for 3 seconds to stop playback, press again to resume.
3. "🌊" Vibration amplitude key: Adjusts the amplitude level. Press the button to change and select the appropriate one from the four values. The default amplitude is 1. Hold the button longer (3 seconds) to disable the function, short press to restart.
4. "🔥" Temperature adjustment button: Press the button briefly to turn off the heating function, the green indicator will flash twice, when pressing again shortly, the heating function will be turned on - the green indicator will flash once. The heating function will work by default when the massager is started.
5. "🔊" Volume key: Increase the volume by one level by pressing the key briefly when the device is on. There are four levels, they will vary

from low to high when pressed. By default, the lowest level starts.

## 7. Description of modes

**Mode 1:** Vibration mode is started automatically, there are eight vibration modes, when selecting the function, the green indicator will flash once.

**Mode 2:** Two-sided intermittent vibration: when this function is selected, the green indicator will flash twice, and the two sides will vibrate at the same time for about 2 seconds with an interval of about 2 seconds between intervals.

**Mode 3:** Wave Type Vibration: After selecting the function, the green indicator will flash three times, the two sides will vibrate at the same time, going from high vibration on one side to low vibration on the other, reminding the wave motion.

**Mode 4:** Two-sided continuous vibration, after selecting the function, the green indicator will flash four times, the vibration strength is stronger.

## 8. Strength of vibrations

**Vibrations can be divided into four stages of force:**

1. The first stage; low, suitable for schoolchildren.
2. second stage; relatively low, suitable for young people.
3. The third stage; moderate, suitable for women and slim people.
4. Fourth degree; the largest, suitable for men and people with high body fat.

## 9. Low power consumption function

The product is set to low power operation by default, when the voltage is too low, the green indicator will flash once every 30 seconds to encourage users to charge the massager battery.

## 10. Charging function

Connect the massager to the computer or adapter using the cable provided in the kit. The battery indicator blinks red while charging. When the battery is fully charged, the indicator light turns red.

## 11. Cleaning and maintenance

- To clean the device, use a soft cloth or a towel soaked in a small amount of clean water, the use of volatile agents is prohibited.
- Store the device in a dry and ventilated place.
- Keep away from direct sunlight, high temperature and moisture.

## 12. Precautions

- Before using this product, please read this manual carefully and fully understand it.
- Clean the band and the massager from any possible dirt.
- People unable to express an opinion may not use this product.
- People with hypersensitive skin or insensitive to heat cannot use this product.
- Pregnant women, people suffering from heart disease, abnormal blood pressure, patients with malignant tumors, cerebrovascular diseases and acute diseases or people taking medications should use

this product under medical supervision.

- If you experience discomfort or have an allergic reaction, stop using the device and consult a physician.
- People with eye diseases: glaucoma, cataracts, retinal detachment or others; or following eye surgery - should use this product under medical supervision.
- Wearers of contact lenses should remove them before use.
- It is forbidden to use this product in high temperature and places that are flammable, wet / humid and exposed to electromagnetic radiation.
- It is forbidden to disassemble this product by yourself, avoid collision, water, otherwise it may breakdown.
- It is forbidden to wash this product with water directly.
- Do not bend or put pressure on the device to avoid damaging the case.

The crossed out garbage can symbol means that in the European Union, after using the product, it should be disposed of at a separate, dedicated point. Do not dispose of these products with unsorted municipal waste.

MEDIVON sp. z o. o. Hereby declares that this massage device complies with the European Directive RE 2014/35/EU The full text of the Declaration of Conformity is available at the following internet address:  
[www.medivon.pl/declarations](http://www.medivon.pl/declarations)

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